FEMMAX® Dilators / Trainers

Providing feminine care
for women of all ages
BPA and Latex free

Indications of Use

The Femmax vaginal dilators/trainers are for use as a manual, reusable device placed into the vagina temporarily to reduce the formation of adhesions after pelvic radiotherapy and to train the vaginal muscles through stretching the vaginal opening when using for psychosexual therapy. The device can also help with other conditions affecting the vulva/vagina.

FEMMAX® Dilators | Trainers
an
MDTI
INNOVATION - INVESTMENT - IMPROVEMENT
product

Version 1.6c Dec 2020

CONTENTS

1. Instructions for use.
2. Assembling the FEMMAX® Dilators / Trainers.
3. Inserting your Dilator / Trainer.
- 3a. Radiotherapy Patients
4. Cleaning Instructions
5. Do's and Don'ts when using a Dilator / Trainer.
6. Additional Notes for radiotherapy patients.
 - 6a. Sexual Intercourse.
 - 6b. Vaginal Discharge.
 - 6c. Menopausal Symptoms.
 - 6d. Vaginal tenderness and discomfort.
7. Other Uses for the FEMMAX® Dilator / Trainer
 - 7a. How to use your trainers/dilators

1. INSTRUCTIONS FOR USE

FEMMAX® vaginal dilators are designed to be safe, easy and convenient to use whilst being able to be stored in a discreet and easily manageable case, which also includes space for a small lubricant bottle to be accommodated.

Vaginal dilators I trainers may be used as part of the care and treatment of women for a number of reasons including treatment following radiotherapy or surgery; difficulties with sexual intercourse; vaginismus; dyspareunia; or following childbirth.

FEMMAX® dilators are designed to be inserted into the vagina to help to keep it healthy and supple and to reduce the risk of adhesions (soft tissue fusion) following radiotherapy or surgery. This may help to reduce discomfort during follow up examinations, further treatments and sexual intercourse.

The dilator set consists of 4 dilators, which are graduated in size and length to allow a natural progression during treatment.

This leaflet offers advice on the use and care of your FEMMAX® dilator set and provides step by step instructions for use.

IMPORTANT: These instructions are intended as a guide and you should seek the advice of your doctor, nurse specialist or therapist before using these dilators.

2. ASSEMBLING THE FEMMAX® DILATORS I TRAINERS

FEMMAX® dilators I trainers are supplied as a set of 4 dilators stored by sliding them into each other and placing them in the accompanying case.

To open the case, press down on the flat area on the front of the case.

To assemble the dilators, lay all 4 dilators down in front of you. At one end of the smallest dilator is a screw thread, which fits into the open end of the next size dilator, allowing the dilators to screw together to become a double ended dilator (see Fig 1).

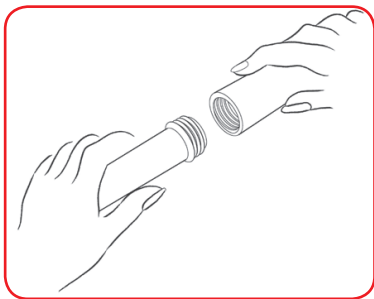
Fig. 1

The two largest dilators also fit together to form a pair (see Fig 1).

One dilator inserts into the vagina and the other becomes a handle to provide support and stability during insertion (see Fig 2).

3. INSERTING YOUR DILATOR / TRAINER

If using the dilators I trainers for the first time, wash them thoroughly in hot soapy water then rinse and dry them to remove all traces of soap (to avoid possible irritation).

Use the dilators in a place that is comfortable, private and allows you to relax as much as possible.

Begin dilation with the smallest size dilator and gradually increase in size over time. Remember you need to stretch the vagina to keep it healthy and supple and you should aim to eventually be using the largest dilator comfortable for you.

To insert the dilator apply a small amount of a vaginal lubricant to the end of the dilator. Lie on your back with your knees bent and slightly apart. If this is not comfortable for you, try standing with one foot on a chair or squatting (similar to inserting a tampon). Try to relax whilst using the dilator (some people find this easier after a bath).

3a. RADIOTHERAPY PATIENTS

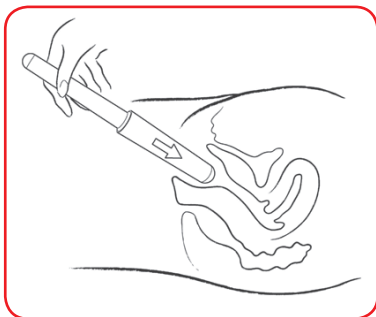
Fig. 2

Put the end of the dilator into your vagina gently and as deeply as possible. Apply a little pressure so that you feel the top of the vagina but without causing discomfort (see Fig. 2).

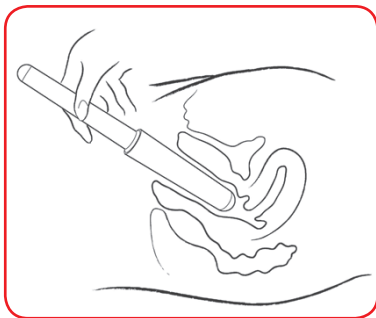
Fig. 3

Hold the dilator firmly and insert to the top of the vagina to a comfortable point, gently rotate the dilator and then withdraw. (Fig. 3).

Some patients may be advised to hold the dilator firmly in place from 5 minutes to 15 minutes, or you may be advised to insert, remove and reinsert, repeating this process several times. Your doctor, nurse or radiographer will advise you fully on this.

Should some spotting or slight bleeding occur after use, this is considered normal because of the breakdown of any adhesions.

However, should heavy or prolonged bleeding occur contact your doctor or nurse specialist immediately.

If you cannot insert the dilator into the vagina easily do not force it.

Remove the dilator and try again later when you feel a little more relaxed.

If after attempting several times you still cannot insert the dilator, stop use and contact your doctor or nurse specialist.

There may be some discomfort when you first start using the dilator. If pain or discomfort continues with use, inform your doctor, nurse specialist or therapist.

It is suggested that a reasonable duration and frequency of dilation may range from three minutes twice a week, for the first six months, up to ten minutes and twice daily. Once a week thereafter and then occasionally after a year if not experiencing difficulty, or as advised by your radiographer, nurse specialist or doctor.

Weekly use is most important even if you continue to have sexual intercourse as it helps to keep the vagina supple and reduces the chances of discomfort during sexual intercourse.

If you are not sexually active it is still advisable to use the dilator as this will assist with medical examination.

4. CLEANING INSTRUCTIONS

This device is supplied Non-Sterile, it is to be cleaned all over with warm soapy water and then dried off before use (wipe dry or air dry) or completely wiped over with a sterile wipe. Before and after each use of the device ensure that you repeat the cleaning instructions. The device can also be cleaned in boiling hot water before use (allow the water to cool before removing). Do not use a disinfectant.

It is recommended that dilators be replaced every 3 years.

5. DO'S AND DON'TS WHEN USING A DILATOR

DO

- Check your dilator for any breaks or cracks or sharp edges before use.
- Contact your doctor or nurse specialist if you have any questions regarding the dilator.
- Try to relax when using the dilator; this will help with insertion.
- Clean your dilator with warm soapy water after each use, checking it carefully.
- Store your dilator in the case provided, this will protect it from damage.
- Use lubrication with the dilator, this will help to ease insertion.
- Find a technique for insertion that is comfortable to you.
- Persevere with using your dilator.
- Warm your dilator in warm water before use if required.

DON'T

- Use your dilator if there are any breaks, cracks, or sharp edges.
- Use your dilator if you experience severe pain or excessive bleeding with use. Contact your doctor or nurse specialist.
- Use any electrical appliances to wash your dilator.
- Store sharp objects in the dilator case as these may damage the dilator.
- Use chemicals or household cleaners on your dilator as these may damage the surface or cause cracking.

6. ADDITIONAL NOTES FOR RADIOTHERAPY PATIENTS

Radiotherapy treatment may have an effect on the lining of the vagina and cause scar tissue (adhesions) to develop and shortening (atrophy) of the vagina to occur. If this happens you may find it difficult or uncomfortable when future internal examination is carried out or when having sexual intercourse.

To help prevent this occurring there are several steps you can take to keep the vagina healthy and supple. Dilation means the stretching and opening of the vagina and sexual intercourse and/or using a dilator can do this.

If you are using your dilator following radiotherapy treatment, it is generally advisable to start using your dilator 2 weeks after your treatment has finished when the acute inflammatory response has settled (but please seek the advice of your doctor or nurse specialist).

6a. SEXUAL INTERCOURSE

Sexual intercourse may continue throughout your radiotherapy treatment. If not, you can resume sexual intercourse when you feel ready, usually at about 2 weeks after the end of the radiotherapy treatment.

It is important to return to your normal level of sexual activity as soon as possible. However, some patients may not feel ready immediately and may feel afraid or even embarrassed, which is not unusual.

There are many different reasons why you may not feel ready for intercourse. You may have lost interest in sex completely, particularly if you feel anxious or tense regarding your health.

Some patients feel nervous about having sex either during or after treatment. It is important to understand that it is perfectly safe to do so.

Cancer cannot be passed to your partner and sex will not make the cancer worse. You may also be concerned about the radiotherapy affecting your partner but radiation does not stay in the body so you will not harm them.

Take your time. Gentle and regular sex will allow you to relax and the vagina to stretch slowly. You may feel more comfortable to start with positions that allow you to be in control (i.e. sitting on top, astride your partner will not only let you set the pace, but will also allow for deep penetration that will assist with dilation). Discuss this with your partner. Show him this leaflet as he may not be aware of your concerns or needs.

If you feel any pain or discomfort and find you have to stop, do not be put off trying again in a few days, as you may simply need a little more time to heal inside. Sometimes if intercourse is difficult, or painful, you may feel tense or anxious. This may also make intercourse more difficult. It is important to try to relax.

If you feel drier than usual, you may need to use a lubricant. There are many vaginal lubricants available to buy from your local chemist.

You may notice slight bleeding on recommencing intercourse. This is due to the tissue stretching and should settle.

Continue using your dilator whether you are sexually active or not as sexual intercourse alone may not be sufficient to prevent adhesions forming.

If you are not sexually active at this time it is still advisable to use a dilator as this will assist with medical examination.

If you feel it would be helpful to talk about any of your concerns contact your doctor or nurse specialist.

6b. VAGINAL DISCHARGE

It is not unusual to have a vaginal discharge after treatment or for it to continue for some time after. However, should the discharge alter in any way, (i.e. become heavier, contains fresh blood or smells unpleasant) contact your nursing team or doctor.

6c. MENOPAUSAL SYMPTOMS

Some women may experience menopausal symptoms, which may be because you have had your ovaries removed during surgery or that you have received radiotherapy treatment.

Radiotherapy treatment may make you infertile. However, you may have a further two or three periods following treatment.

Therefore, there is still a risk of pregnancy and you should continue with your normal method of contraception until advised otherwise. You may be concerned about the effect of the menopause brought on by the treatment. Some patients experience hot flushes, feeling low in mood, tension, anxiety, irritability, vaginal dryness, loss of sex drive and possible weight gain. Do talk to your GP or doctor at your hospital check-up, as treatment and help may be available.

6d. VAGINAL TENDERNESS AND DISCOMFORT

- Some women have redness, tenderness or discomfort to the skin around and inside the vagina during or after radiotherapy. This may be eased by several methods:
- Keep the area clean and dry.
- Wash only with water.
- Wear cotton underwear.
- Use only a recommended vaginal lubrication for intercourse and when using the dilator.
- If symptoms persist contact your doctor or nurse specialist.

7. OTHER USES FOR THE FEMMAX DILATOR I TRAINER

The Femmax vaginal dilators can also be used during psychosexual therapy (sexual difficulties/vaginismus), by women with Lichen Sclerosus, Vulval Lichen Planus, dyspareunia, vulvodynia, vulvar vestibulitis, menopause, vaginal atrophy, after female gender reassignment surgery and for strengthening the pelvic floor, as advised by a clinician.

7a. HOW TO USE YOUR TRAINERS/DILATORS

1. Use the trainers in a place that is comfortable and ensures your privacy.
2. The recommended position to use the trainers is to lie flat on your back with your knees bent and legs slightly apart. Alternatively stand with one leg raised on a chair.
3. Starting with the smallest trainer, apply the lubricant around the tip of the trainer and a small amount to your vaginal opening.
4. Controlling your breathing, relaxing as much as possible gently ease the trainer into your vagina as deeply as is comfortable and leave it in position for up to 5 minutes.
5. You may find when you begin using the trainers that you can only tolerate the tip of the trainer inside your vagina. This is perfectly normal.
6. This sequence should be repeated daily.
7. When you feel comfortable with using the smallest trainer, gradually move to the next size and so on.
8. There is no time limit on the time it takes to progress to each new size. It is advised that when you can comfortably insert the smallest size trainer, change to the next size in the same session, as your vaginal muscles will already be relaxed and you will find it easier to progress to the next size.
9. This also applies as you progress to the other sizes. When you feel comfortable with inserting the larger trainers you may be ready to attempt penetrative sexual intercourse.

Instructions courtesy of Mary Clegg, an Accredited Sex and Relationship Psychotherapist www.maryclegg.co.uk

FEMMAX® Dilators I Trainers

an

product

More information & **instructional videos**
about the Femmax are available at
www.mdti.co.uk/femmax

FEMMAX® Dilators / Trainers
- THE SIMPLE EFFECTIVE SOLUTION
AN NHS INNOVATION

Enhancing safety and efficiency

EU AR: MDTi Cyprus Ltd, 1 Kalymnou Q MERITO,
4 floor, 6037 Larnaca, Agios Nikolaos, Kamares, Cyprus.
E-mail: info@mdti.eu

MDTi, The Kace Building, Victoria Passage,
Wolverhampton, West Midlands,
WV1 4LG United Kingdom.

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