

## **HOOK-IT RANGE INSTRUCTIONS FOR USE**



HOOK-IT™ J-HOOK

Lilac 15mm, Turquoise 21mm.

Used to suspend IV and fluid bags from a wall bracket or 15mm/21mm width curtain track. J-HOOK will take up to two IV bag, max load 5kgs, (will break under human weight, anti-ligature).

The hook can be cleaned with soap and water or alcohol. The J-HOOK can withstand multiple sterilisation cycles carried out at 135°.

**When removing the J-HOOK from the curtain track please ensure J-Hook is LIFTED directly UP from track and NOT removed at an angle.**



HOOK-IT™ Wall Bracket

Make sure that the text on the front of the bracket is upright.

The wall bracket should be mounted approx. 5.5 ft on a wall from ground level. Position a bracket either side of a patient's bed if space allows.

The bracket can be cleaned with soap and water or alcohol.



HOOK-IT™ C-HOOK

Two C-HOOKS are to be fitted to the side rails of a bed frame in order to carry a catheter or drainage bag.

The hook can be cleaned with soap and water or alcohol.

The C-HOOK can withstand multiple sterilisation cycles carried out at 135°.



HOOK-IT™ B-HOOK

Fits on the head of the frame to carry fluid bags when transporting patients on beds.

The hook can be cleaned with soap and water or alcohol.

The B-HOOK can withstand multiple sterilisation cycles carried out at 135°.

[For technical support and customer services please contact;](#)

**Medical Devices Technology International Ltd (MDTi)**

Registered Office: The Kace Building, Victoria Passage, Wolverhampton, West Midlands WV1 4LG, United Kingdom

Direct Line No. +44 (0) 1902 778380

Company No. 04383577

Vat No. 836853102

LRQ Approval Certificate No.4003482/A

LRQ Approval Certificate No.4004298

*MDTi commercialises products by the NHS for the NHS*