TFK95/Med Rev 04

User Instructions

INFORMATION FOR USERS

Product: MODEL KN95(NR) /FFP2 PARTICULATE RESPIRATOR

Model No.: 4020

Manufacturer: Medical Devices Technology International Ltd, The Kace Building, Victoria Passage,

Wolverhampton WV1 4LG, West Midlands United Kingdom.

CE Approvals: The product meets the requirements of the European PPE Regulation (EU) 2016/425 and carry the CE mark. Certification under Module B CE 730967, EU Type-Examination and Module C2 CE 730969, EC Quality Control, has been issued by BSI, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes MK5 8PP, UK (Notified Body Number 0086).

RISK THAT PPE IS DESIGN TO PROTECT: Designed to support the use of disposable respirators as part of the PPE ensemble worn by Healthcare Professionals for reduction of aerosol generating around patients.

WARNING:

This filtering half mask is manufactured for COVID-protection only. This filtering half mask is not a PPE device for general use and shall not be used for purposes other than protection of COVID-19.

This product is a single user disposable product, only used for respiratory protection of some particulate matter.

Fitting reference is shown below. If you cannot achieve proper fit, do not enter a contaminated area.









HOW TO USE A MASK

- Before wearing a mask, wash your hands with an alcohol-based disinfectant or with soap and warm water.
- Cover your mouth and nose with the mask and make sure the mask is firmly pressed against your face.
- Do not touch the mask while you are wearing it; if you do, wash your hands with an alcohol-based disinfectant or with soap and warm water afterward.
- Replace the mask as soon as it gets wet and do not reuse disposable masks.
- Remove the mask from behind (do not touch its front side); throw it away in a closed container and then wash your hands with an alcohol-based disinfectant or with soap and warm water.

Storage: Store masks in a dry, ventilated, and non-corrosive gas environment. **Ideal temperature:** 0 - 50° C, relative humidity: below 50% RH, **Shelf Life:** 2 year

Copy: Master Issue: 1.4

Date: 07/09/2020

TFK95/Med Rev 04

Copy: Master Issue: 1.4

Date: 07/09/2020