

HOOK-IT RANGE INSTRUCTIONS FOR USE



HOOK-IT™ J-HOOK

Blue 15mm (FKW010) and 21mm (FKW011)

Used to suspend IV and fluid bags from a wall bracket or 15mm/21mm width curtain track. J-HOOK will take up to two IV bag, max load 5kgs, (will break under human weight, anti-ligature).

The hook can be cleaned with soap and water or disinfectant wipe as recommended by the Trust Infection Control Policy. The J-HOOK can withstand multiple sterilisation cycles carried out at 135°.

When removing the J-HOOK from the curtain track please ensure J-Hook is LIFTED directly UP from track and NOT removed at an angle.



HOOK-IT™ Wall Bracket

Make sure that the text on the front of the bracket is upright.

The wall bracket (FKW016) should be mounted approx. 5.5 ft on a wall from ground level. Position a bracket either side of a patient's bed if space allows and, in the bathroom/toilet.

The bracket can be cleaned with soap and water or disinfectant wipe as recommended by the Trust Infection Control Policy

For technical support and customer services please contact;

Medical Devices Technology International Ltd (MDTi)

Registered Office: The Kace Building, Victoria Passage, Wolverhampton,
West Midlands WV1 4LG, United Kingdom

Direct Line No. +44 (0) 1902 778380

Email : info@medti.co.uk

Company No. 04383577

Vat No. 836853102

European Authorized Representative: MDTi Cyprus Ltd, 1 Kalymnou Q MERITO, 4 floor, 6037 Larnaca, Agios Nikolaos, Kamares, Cyprus. Telephone No. +357 24343145, Email: info@mdti.eu

MDTi commercialises products by the NHS for the NHS

